

Using a G3 defibrillator



- 1** Follow the steps below if the patient is unconscious, not breathing, or breathing irregularly
- 2** Call the service center at 08-934-2999
- 3** **Start CPR:**
perform chest compressions at a rate of 100/minute
- 4** **Turn on the defibrillator:**
press the yellow button and open the gray cover
- 5** Expose the patient's chest
- 6** Open the electrode packages that can be found in the inner part of the gray cover
- 7** Remove the safety ribbon and attach the stickers to the patient's chest according to the diagram on the stickers
- 8** If the device recommends a "shocking" the patient with the device, announce "shocking patient, stand clear"
- 9** press the blinking red "shock" button as per the directions on the device
- 10** **Immediately continue chest massage and CPR and follow the instructions on the defibrillator:**
every 2 minutes the defibrillator will check if the patient needs another "shock"

Using a defibrillator HeartSine

- 1** Follow the steps below if the patient is unconscious, not breathing, or breathing irregularly
- 2** Call the service center at 08-934-2999
- 3** **Start CPR**
Perform chest compressions at a rate of 100/minute
- 4** Turn on the defibrillator by pressing the green button
- 5** Expose the patient's chest
- 6** Pull the green tab and take out the sticker tray
- 7** Remove the safety ribbon and attach the stickers to the patient's chest according to the diagram on the stickers
- 8** If the device recommends a "shocking" the patient with the device, announce "shocking patient, stand clear"
- 9** Press the orange "shock" button as per the directions on the device
- 10** **Immediately continue chest massage and CPR and follow the instructions on the defibrillator**
every 2 minutes the defibrillator will check if the patient needs another "shock"

